

## BIOMAG-I

A new resorbable magnesium scaffold for de novo coronary lesions (DREAMS 3G): 12-month results of the BIOMAG-I first-in-human study<sup>1</sup>

### Conclusions

- DREAMS 3G has an improved angiographic in-scaffold LLL compared to its precursor DREAMS 2G<sup>2</sup> at 12-month follow-up, making DREAMS 3G a potential alternative to permanent DES.<sup>3</sup>
- At 12-month follow-up, DREAMS 3G shows low TLF (2.6%) and clinically-driven TLR rate (2.6%), no myocardial infarction and no definite or probable scaffold thrombosis.
- Furthermore, at 12-month follow-up, patients treated with DREAMS 3G demonstrated a return of vasomotion.<sup>4</sup>
- OCT analysis revealed that 99.3% of struts were no longer visible at 12 months confirming the preclinical data.<sup>5</sup>

### Study design

- Prospective, multi-center, single arm
- Subjects with single de novo coronary artery lesions in up to two coronary arteries
- Reference Vessel Diameter (RVD) between 2.5-4.2 mm and lesion length  $\leq$  28 mm long

### Patients

20.7% of NSTEMI patients included

### Endpoints

#### Primary endpoint

- In-Scaffold Late Lumen Loss (LLL) at 6 months

#### Angiographic Endpoints (at 6 and 12 months)


- In-Segment LLL
- Binary restenosis rate
- % In-Scaffold and In-Segment Diameter Stenosis
- In-Scaffold LLL (powered for both time points)

#### Clinical Endpoints

- Target Lesion Failure\* (TLF)
- Definite or Probable Scaffold Thrombosis

#### Imaging and Physiological Endpoint

- Procedure and device success
- Descriptive analysis of vessel morphology, lesion composition and scaffold strut data (IVUS and OCT)
- Descriptive analysis of vasomotion

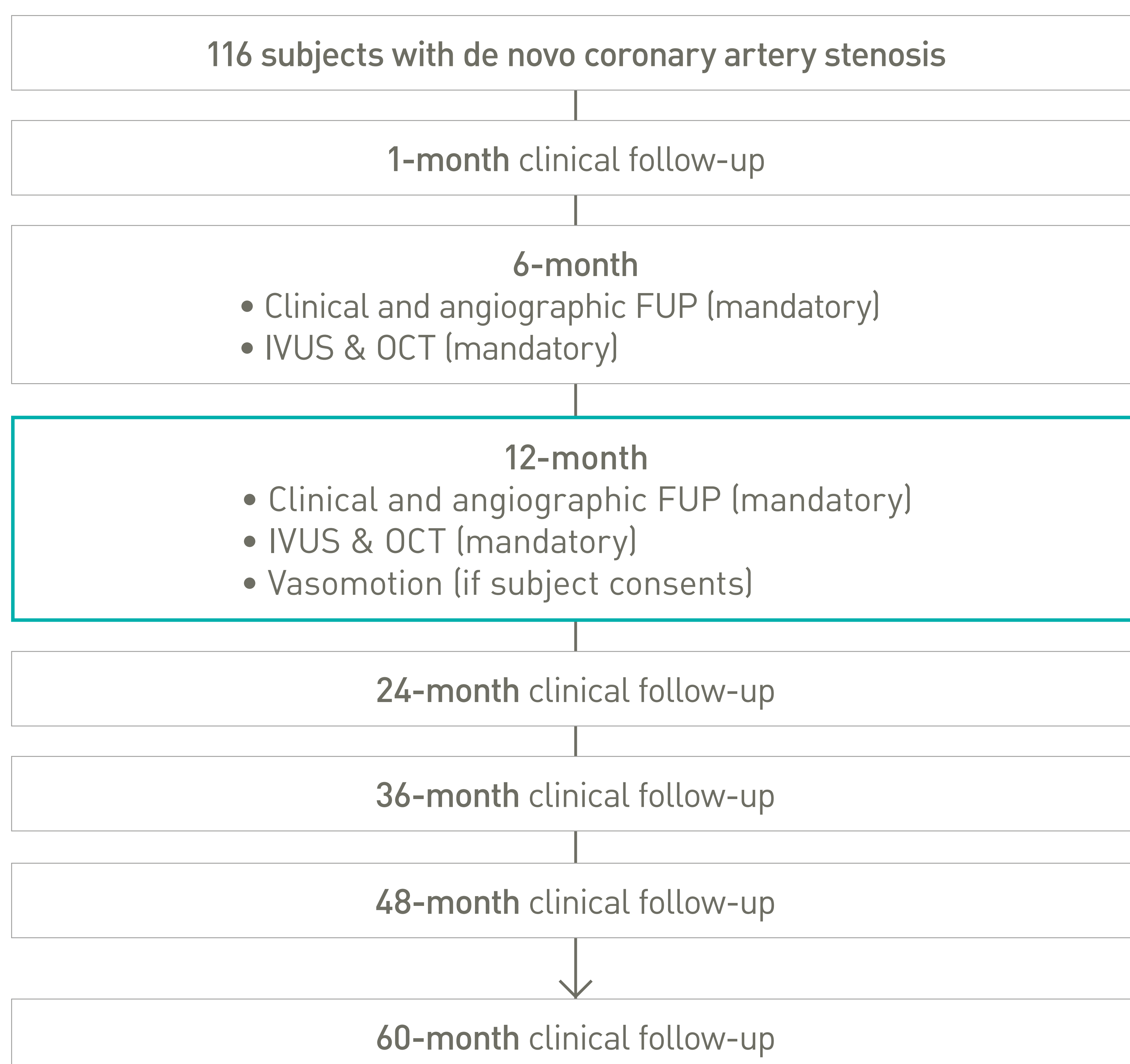


Patient characteristics	n = 116	%
Age, years	61.0 ± 9	
Male	90	77.6%
Hypertension	86	74.1%
Hypercholesterolemia	72	62.1%
Diabetes	32	27.6%
History of smoking	75	64.7%
History of myocardial infarction	39	33.6%
<b>NSTEMI</b>	<b>24</b>	<b>20.7%</b>

Lesion location	n	%
LAD	53	45.3%
LCx	22	18.8%
RCA	40	34.2%
Ramus intermedius	2	1.7%


Lesion characteristics	n	%
Lesion length (mm)	12.3 ± 5.1	
Reference vessel diameter (mm)	2.72 ± 0.46	
<b>AHA/ACC lesion class B2/C</b>	<b>90</b>	<b>76.9%</b>
Side branch involvement	25	21.4%

Lesion characteristics are estimated by core lab.

## In-Scaffold Late Lumen Loss at 12 months<sup>1</sup> (n = 100)

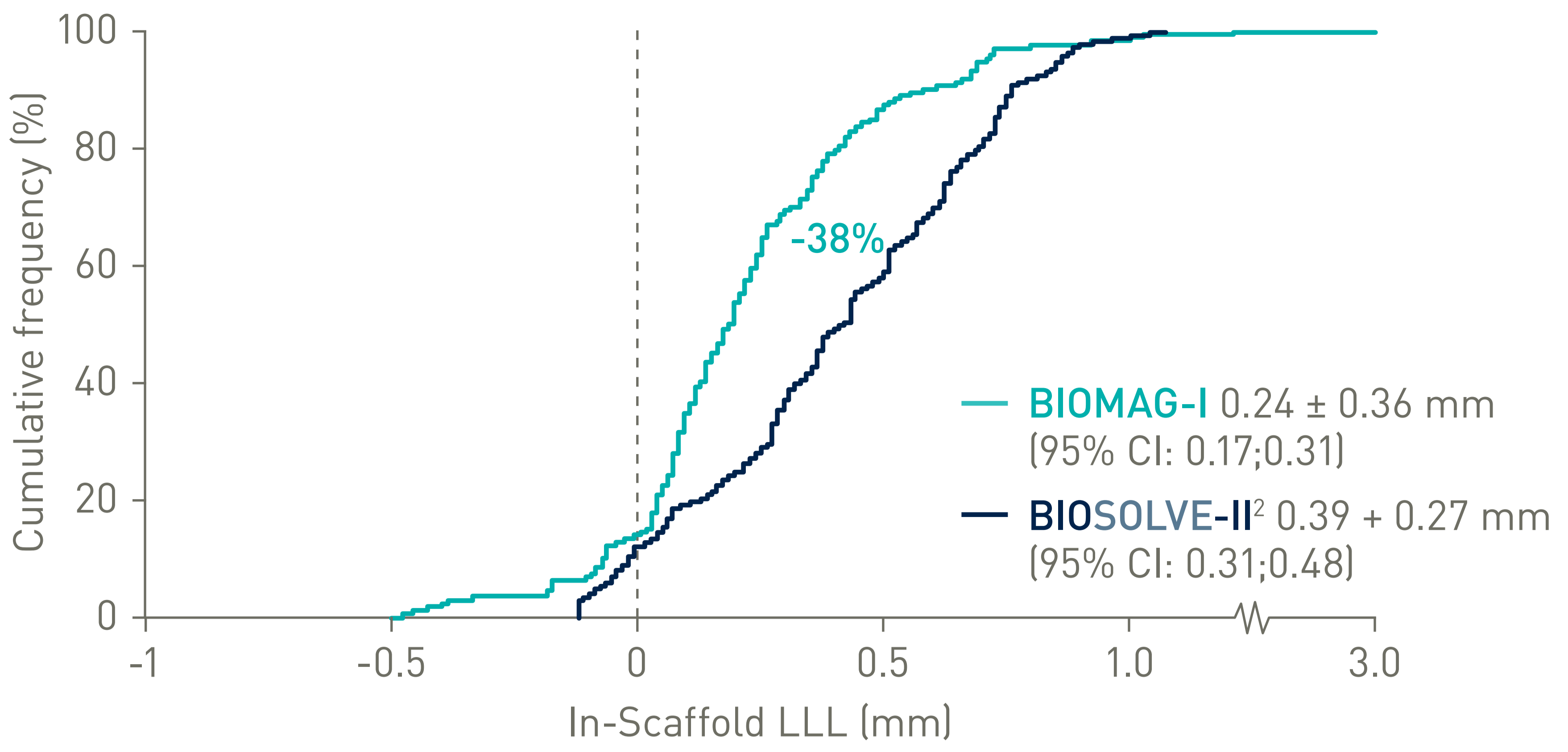


## In-Segment Late Lumen Loss at 12 months<sup>1</sup> (n = 100)

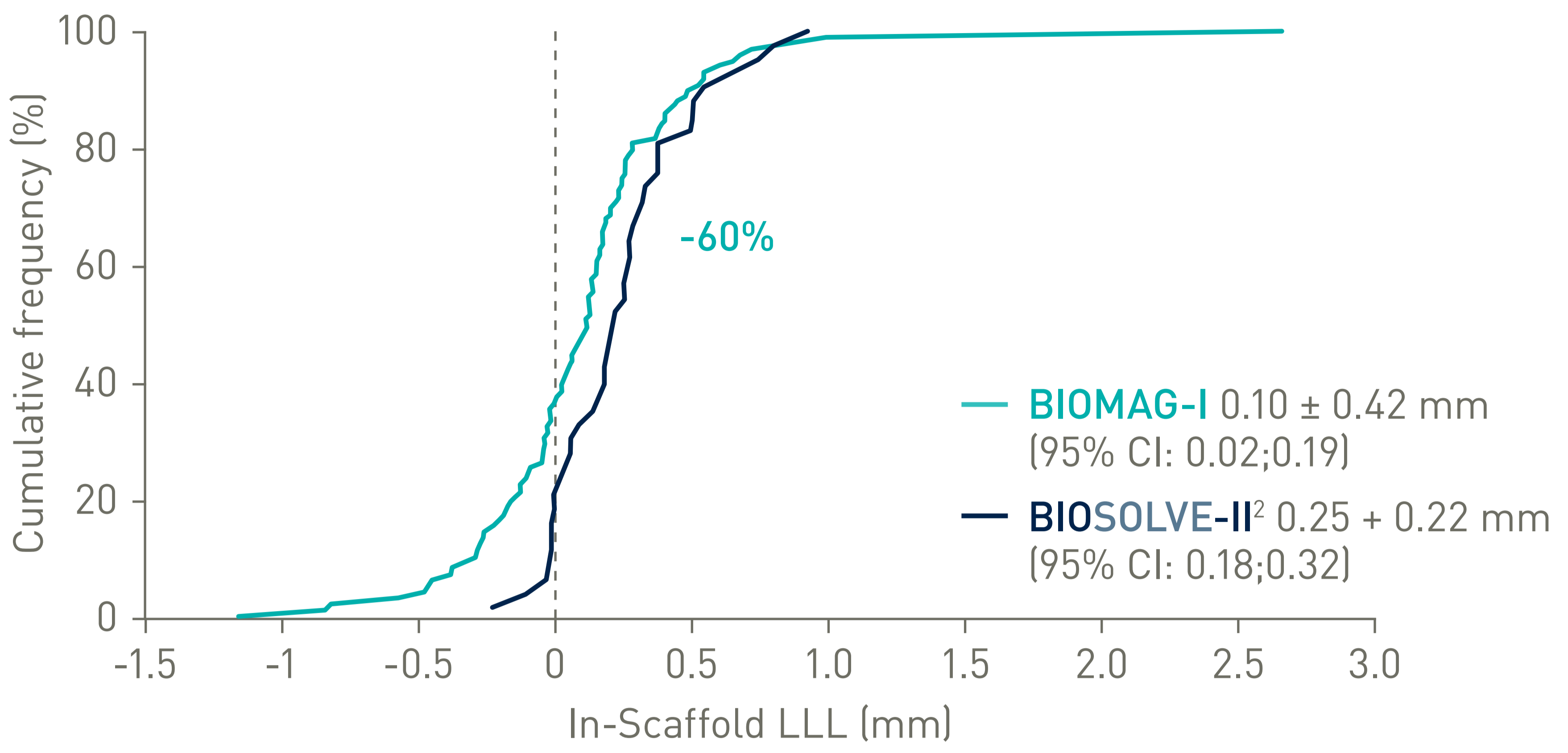

## Comparison of the clinical outcomes at 6 and 12 months

	BIOMAG-I 6m n = 116	BIOMAG-I 12m n = 116	BIOSOLVE-II <sup>2</sup> 12m n = 121
Target lesion failure*	0.9%	2.6% <sup>o</sup>	3.4%
Cardiac death	0.0%	0.0%	0.8%
TV-MI**	0.0%	0.0%	0.8%
CD-TLR	0.9%	2.6%	1.7%
Definite or probable ST	0.0%	0.0%	0.0%

## Serial OCT and IVUS Data Analysis

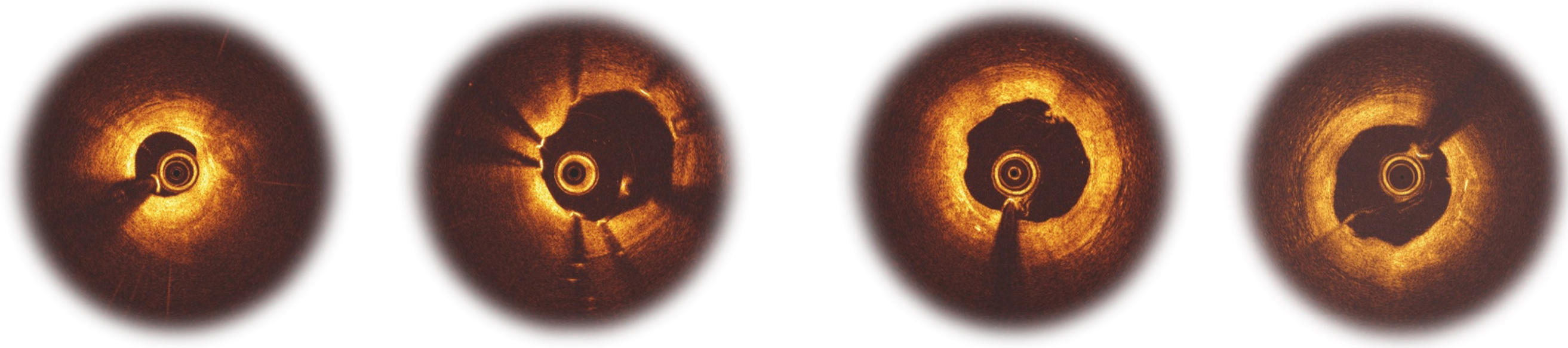

Pre-procedure

Post-procedure

6m follow-up

12m follow-up

- Malapposed<sup>†</sup> struts %: 4.62 ± 4.69
- Total incomplete strut apposition area mm<sup>2</sup>: 0.08 ± 0.11

Struts hardly discernable

Struts were not discernable anymore

	Baseline	Post-procedure	6-month	12-month
Mean lumen area (mm <sup>2</sup> )	5.96 ± 2.25	7.65 ± 2.23	7.00 ± 2.42	7.09 ± 2.73
				Δ Post-procedure - 12m
				-0.56 ± 1.45 <sup>‡</sup>

## Coordinating investigator

Michael Haude, MD, PhD, Rheinland Klinikum, Neuss, Germany

\*TLF is defined as Composite of Cardiac Death, TV-MI, CD-TLR (Kaplan-Meier estimate); \*\*peri-procedural target vessel MI according to SCAI definition and non-peri-procedural target vessel MI according to Universal MI Definition; <sup>o</sup>driven by three clinically-driven target lesion revascularization; <sup>†</sup>Definition of malapposition: if the distance between outer contour of the strut and vessel wall is more than the individual strut thickness; <sup>‡</sup>p < 0.05 for 12-month vs post-procedure. 1. Haude, M "1-Year Clinical Outcomes of the new resorbable Magnesium scaffold DREAMS 3G, from the first in-human BIOMAG-I study" presented at EuroPCR May 2024; 2. Haude M, et al., Sustained safety and performance of the second-generation drug-eluting absorbable metal scaffold in patients with de novo coronary lesions: 12-month clinical results and angiographic findings of the BIOSOLVE-II first-in-man trial. Eur Heart J 2016;37:2701-2709; 3. Byrne RA, et al., Report of a European Society of Cardiology-European Association of Percutaneous Cardiovascular Interventions task force on the evaluation of coronary stents in Europe: executive summary. Eur Heart J 2015;36:2608-262; 4. Haude, M "First in human study BIOMAG-I: 12 months results of the sirolimus eluting resorbable coronary magnesium scaffold system (DREAMS 3G) in the treatment of subjects with de novo coronary artery lesions" presented at ESC August 2023; 5. Seguchi, M "Twelve-months vessel healing profile following the novel resorbable magnesium scaffold implantation: an intravascular OCT analysis of the BIOMAG-I trial" presented at ESC August 2023.

All endpoint related events have been adjudicated by an independent clinical event committee. BIOMAG-I and BIOSOLVE-II are based on Kaplan-Meier failure estimate analysis including censored observations. DREAMS 3G next generation Resorbable Magnesium Scaffold is not CE or FDA approved and not commercially available.

Magmaris, BIOMAG und DREAMS 3G are trademarks or registered trademarks of the BIOTRONIK Group of Companies. DREAMS 3G next generation Resorbable Magnesium Scaffold is not CE or FDA approved and not commercially available.