

1


Timing Cycles: Single Chamber

- There is one basic interval connecting 2 consecutive events, pacing or sensing: AA or VV.
- There are two basic transitions, or pacing events that cause the basic interval to be reset (reinitiated).
- If the interval times out, a stimulus is released, and the interval starts over.
- If a spontaneous event is sensed, the interval is reset without pacing, the pending stimulus being "inhibited."

2

2

VVI

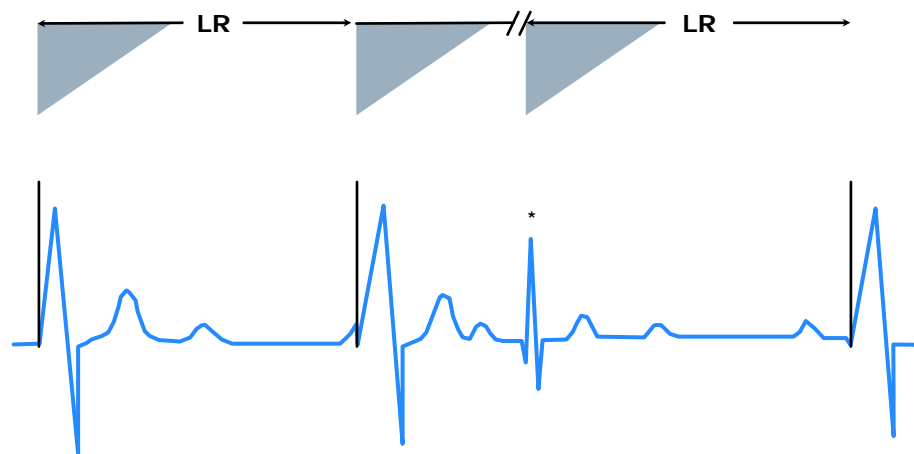


3

BIOTRONIK
excellence for life

3

VVIR

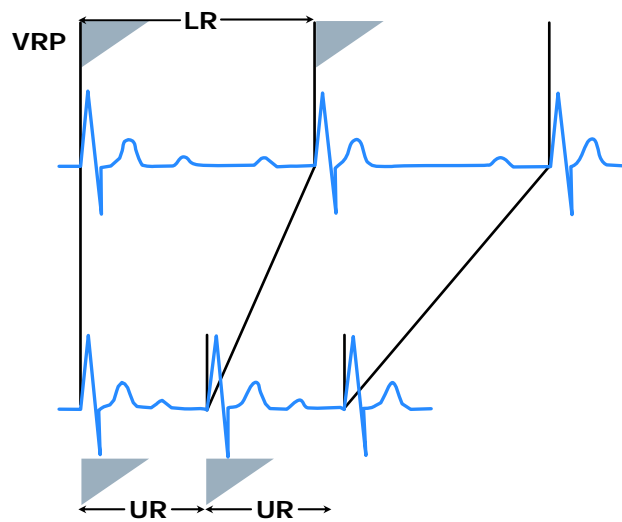

4

BIOTRONIK
excellence for life

4

Timing Cycles: Dual Chamber

- Two basic intervals: AV & VA; their sum is the cycle duration or “cycle length” AA or VV.
- The DDD mode has five basic transitions, each of which resets (re-initiates) either the AV or the VA interval depending upon the event that initiates the transition.

5



5

Five Transitions of DDD Timing

Event

AV interval times out


Response

Pace V, begin VAI

6



6

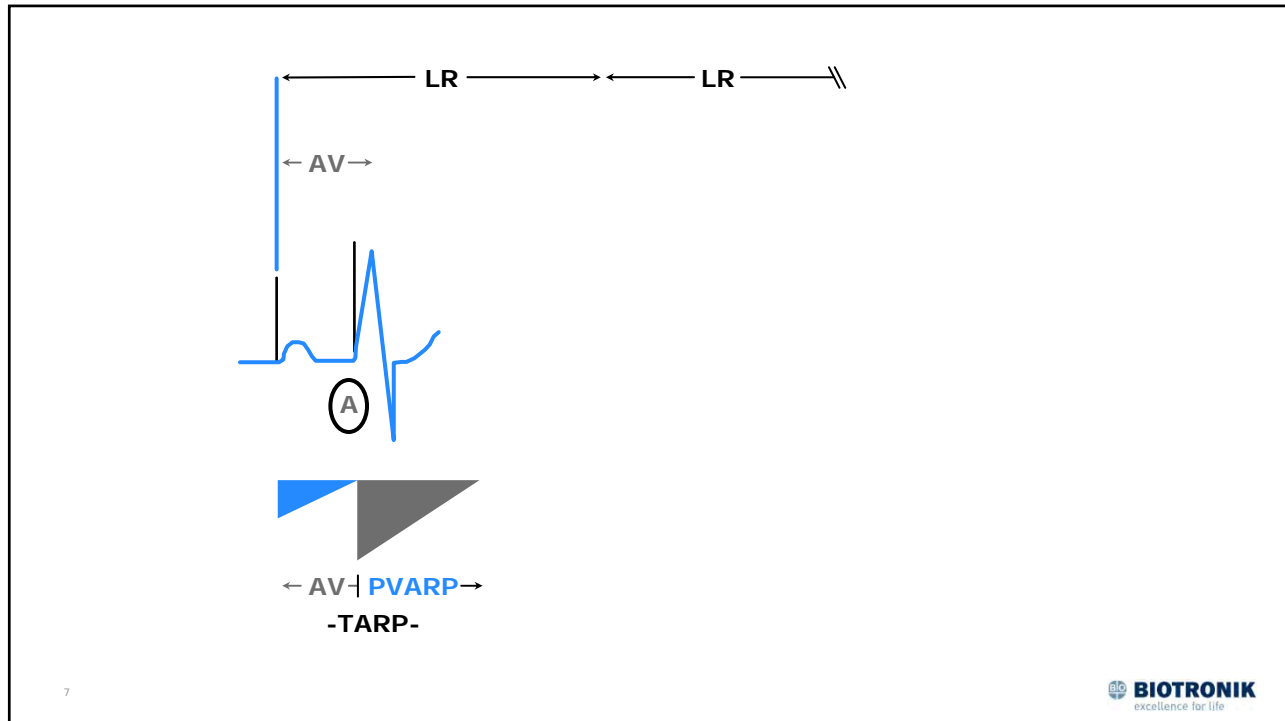

7

Five Transitions of DDD Timing

Event

AV interval times out

VA interval times out


Response

Pace V, begin VAI

Pace A, begin AVI

8

8

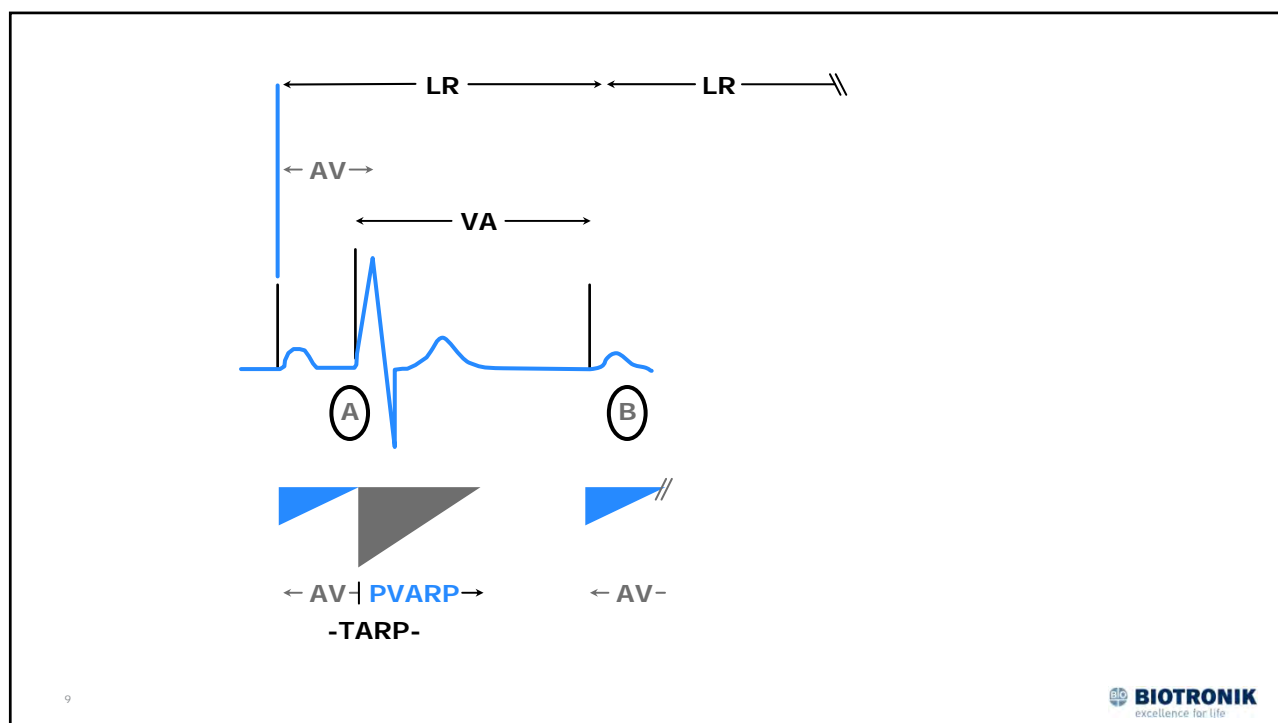

9

Five Transitions of DDD Timing

Event


AV interval times out
 VA interval times out
 V sense during AVI
 V sense during VAI

Response

Pace V, begin VAI
 Pace A, begin AVI
 Begin VA/no pacing
 Re-start VA/no pacing

10

10

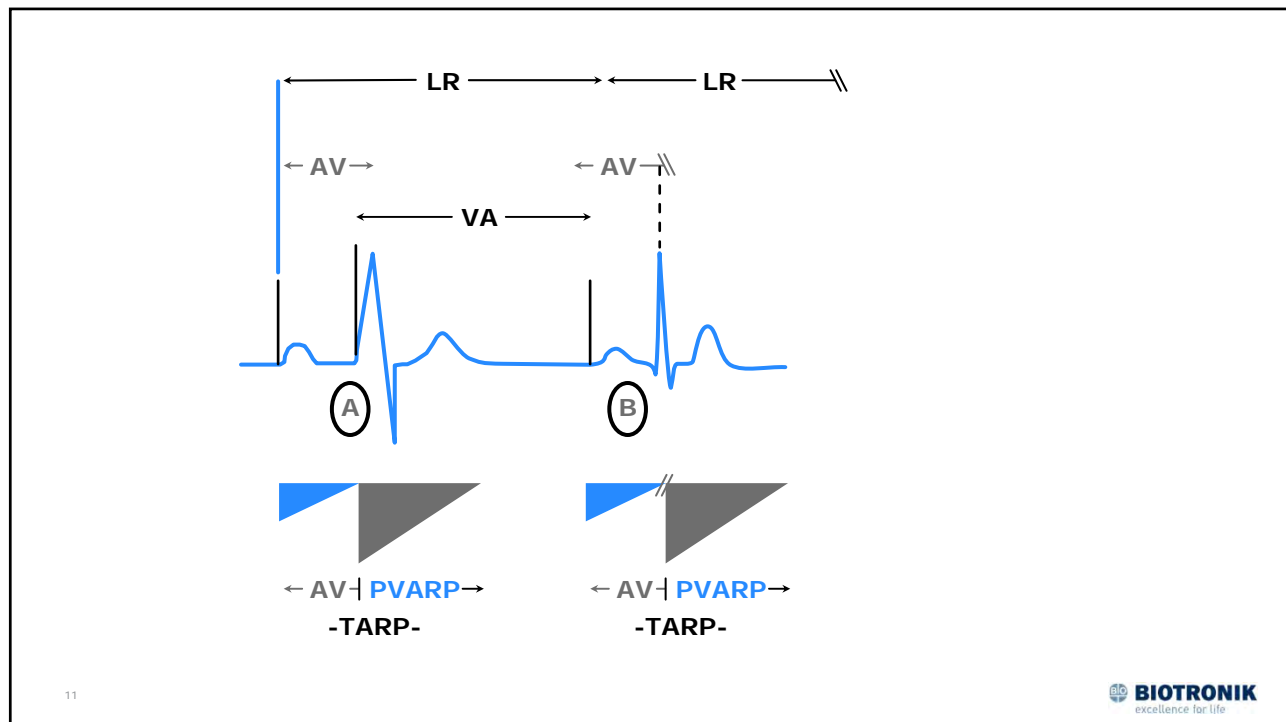

11

Five Transitions of DDD Timing

Event


AV interval times out
 VA interval times out
 V sense during AVI
 V sense during VAI
 A sense during VAI

Response

Pace V, begin VAI
 Pace A, begin AVI
 Begin VA/no pacing
 Re-start VA/no pacing
 Begin AVI/no pacing

12

12

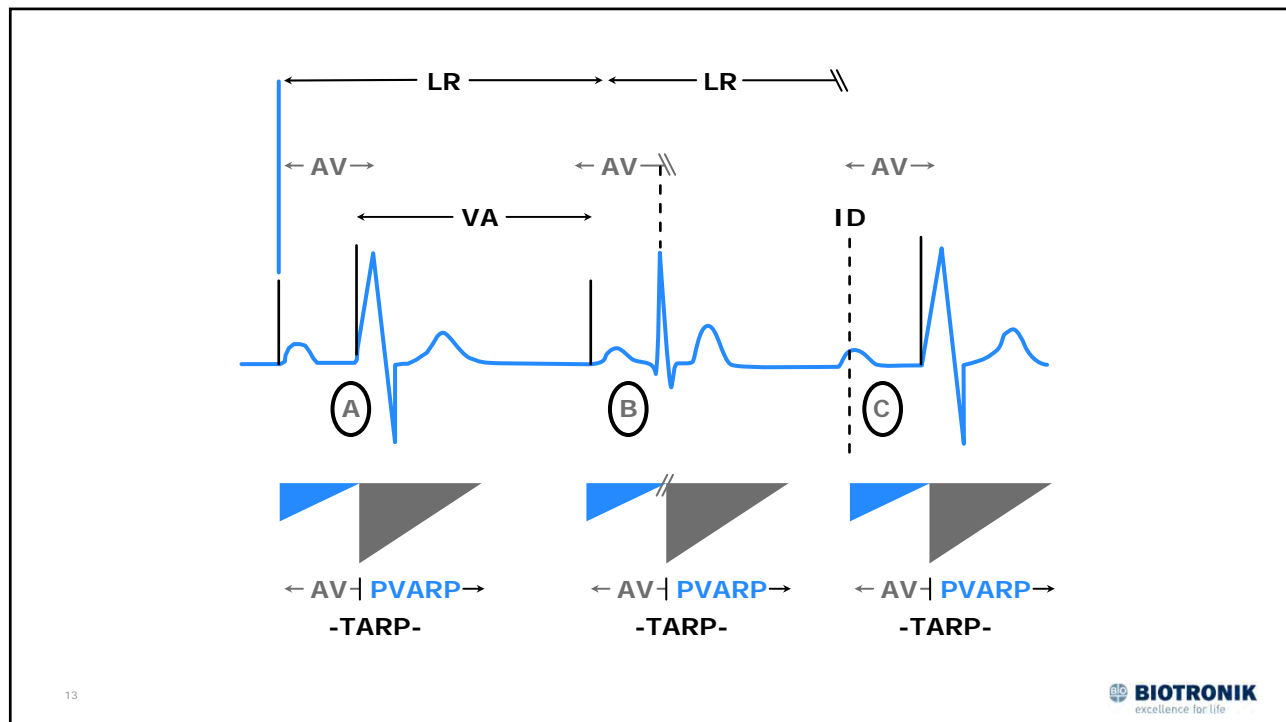

13

Which of the following defines the maximum tracking rate of a DDD pacemaker?

1. PVARP
2. VRP + AVI
3. TARP
4. AVI + Blanking period
5. TARP - VRP

14

TARP = AVI + PVARP


- AVI = 200 ms
- PVARP = 400 ms
- TARP = 600 ms = 100 bpm

TARP = Total Atrial Refractory Period

PVARP = Post-ventricular atrial refractory period

15

15

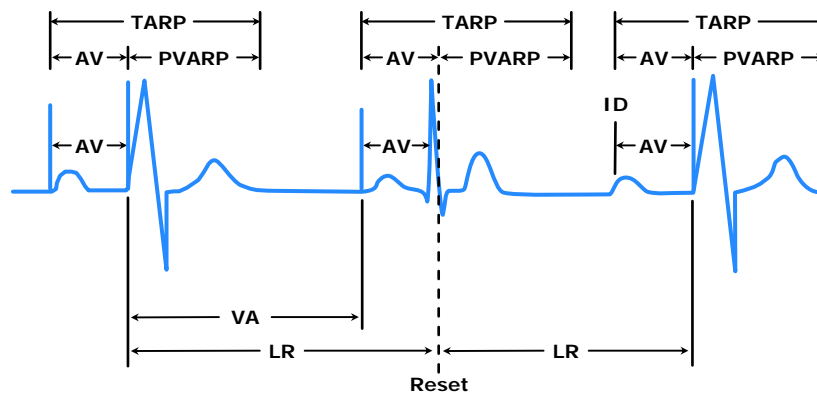

16

16

Timing Cycles


- During a refractory period, sensed events are ignored for timing purposes – although they would usually be annotated on 'marker' channel.
- During a blanking period, the sensing amplifier is turned off and sensing cannot take place.
- The pattern of refractory and blanking periods plays a critical role in determining the beat-to-beat behavior of the pacemaker.

17

17

If the PM represented in this schematic is functioning normally, what is the pacing mode?

1. DDD
2. AAI
3. VVI
4. VDD
5. VAT

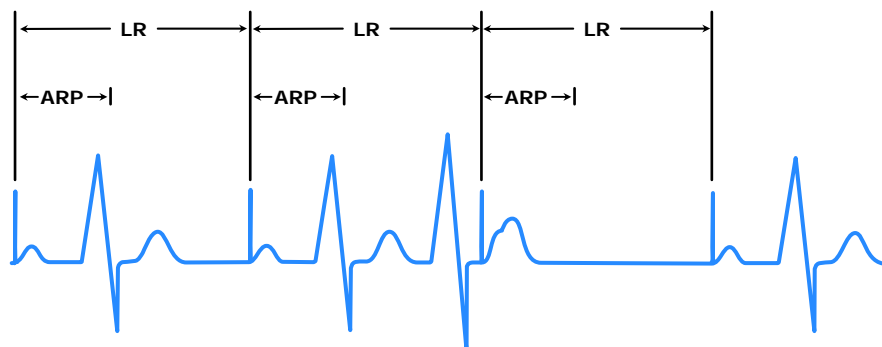


18

18

If the PM represented in this schematic is functioning normally, what is the pacing mode?

1. DDD
2. AAI
3. VVI
4. VDD
5. DDI

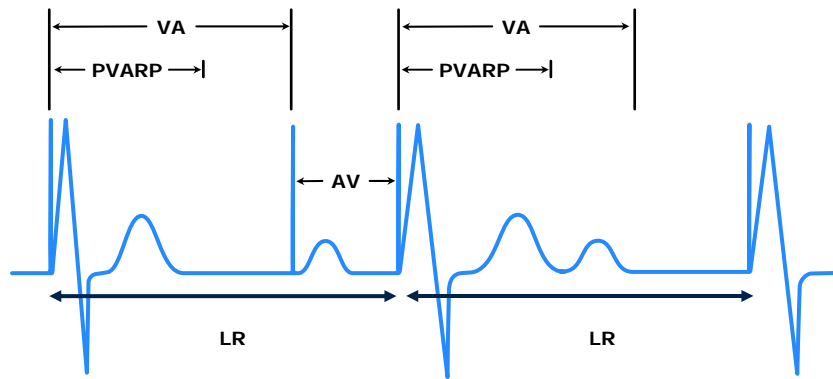


19

BIOTRONIK
excellence for life

19

If the PM represented in this schematic is functioning normally, what is the pacing mode?

1. DDD
2. AAI
3. VVI
4. VDD
5. VAT

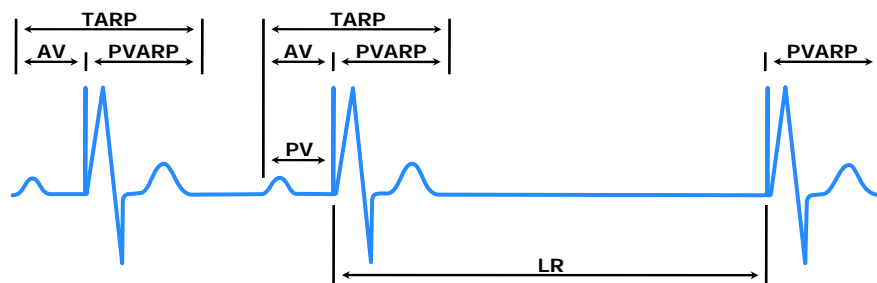


20

BIOTRONIK
excellence for life

20

The Timing Cycle Is Compatible With:

1. AAI with farfield sensing
2. VDD pacing mode
3. Normal blanking
4. Normal VVI

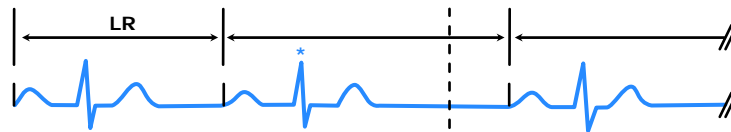


21

 **BIOTRONIK**
excellence for life

21

The Timing Cycle Is Compatible With:

1. **AAI with farfield sensing**
2. VDD pacing mode
3. Normal blanking
4. Normal VVI

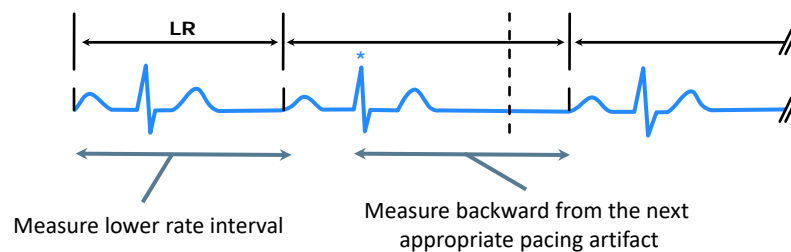

22

 **BIOTRONIK**
excellence for life

22

Specific Timing Cycles

- AAI or AAIR and appropriate lack of response to ventricular events
- VDD or VDDR and reversion to VVI timing in the absence of atrial activity
- DDI or DDIR and the lack of atrial tracking (*in DDI--lower rate = upper rate*)

23



23

Timing Cycles: Crosstalk

- Unwanted detection in one channel of a signal from another channel
- Most common: afterpotential from atrial output sensed by V channel and resets VA interval
- AV crosstalk easier to avoid with bipolar sensing configuration, less sensitive to far-field signals


24



24

If the event that occurs in the crosstalk sensing window is sensed, the V is paced after an abbreviated AVI (light orange), falling in the physiologic refractory period; i.e., does not fall in the vulnerable period.

If the event occurs in the post-atrial ventricular blanking period, the sensing circuit is turned "off" and the event is not seen (dark orange).

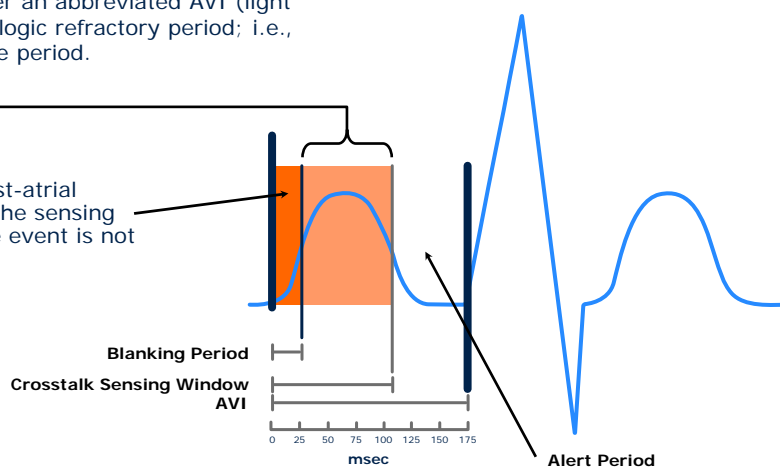


If sensing occurs during the 'alert' period, ventricular output is inhibited.

25

BIOTRONIK
excellence for life

25

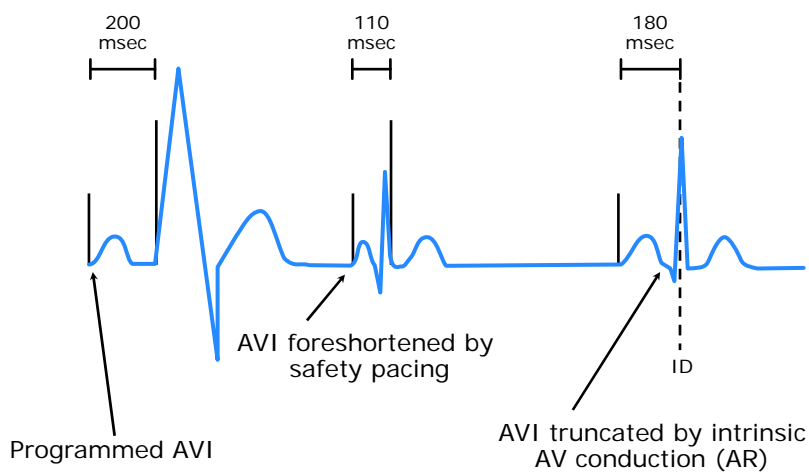

26

BIOTRONIK
excellence for life

26

Timing Cycles: Crosstalk

AV crosstalk may be eliminated by:


- Increasing the blanking period *(so that ventricular sensing resumes later)*
- Decreasing the atrial stimulus voltage or duration *(if adequate safety margin can be maintained)*
- Making ventricular channel less sensitive

27

27

Programmed AV = 220; The labeled QRS complex (*) occurs in the:

1. Crosstalk sensing window
2. Alert portion of AVI
3. Post-atrial ventricular blanking period

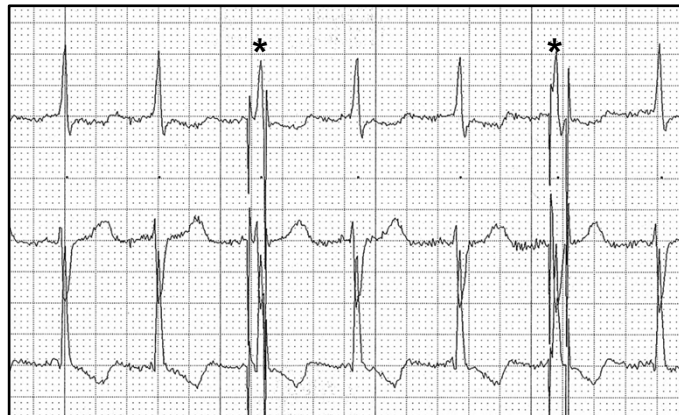


28

28

Programmed AV = 220; The labeled QRS complex (*) occurs in the:

1. Crosstalk sensing window
2. Alert portion of AVI
3. Post-atrial ventricular blanking period

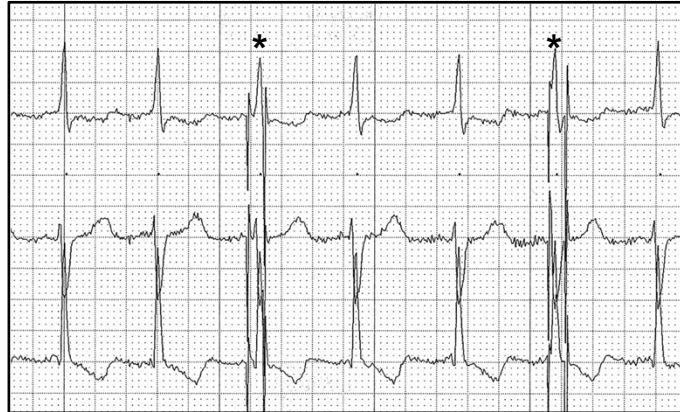

29

 **BIOTRONIK**
excellence for life

29

Upper Rate Behavior

Components of basic timing


- TARP (AVI + PVARP)
- Dynamic AVI
- Rate-adaptive parameters
- Timing system

30

 **BIOTRONIK**
excellence for life

30


Schematic of DDD Timing

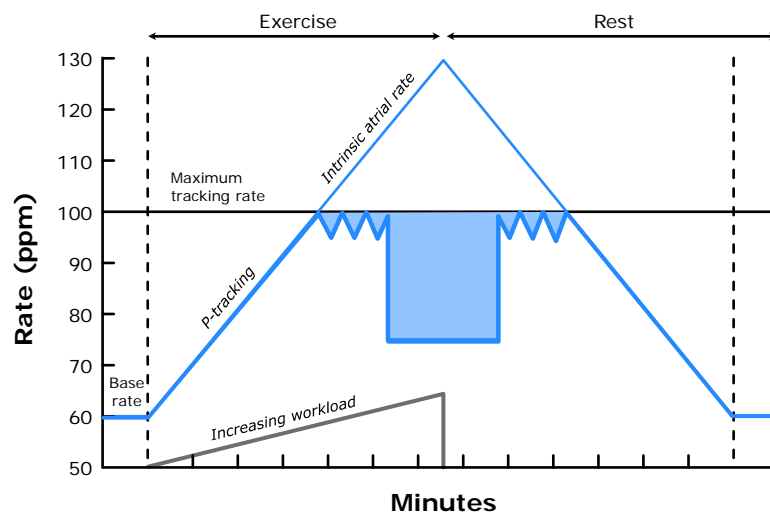

31

31


DDD Upper Rate Behavior

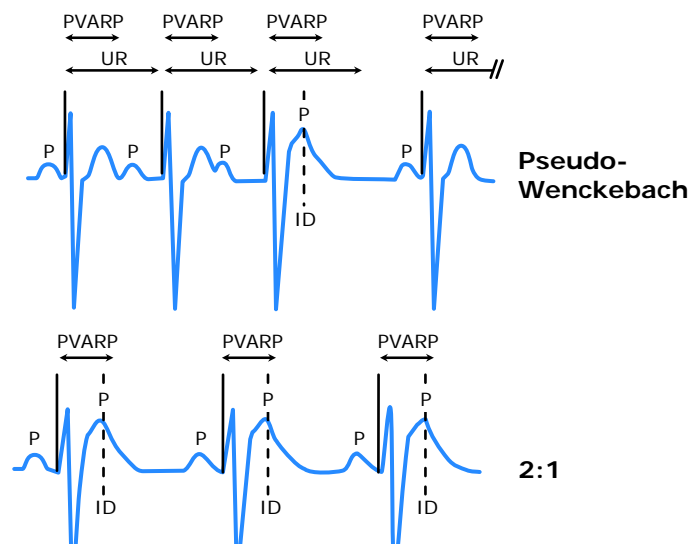

32

32

Schematic of DDD Upper Rate Behavior

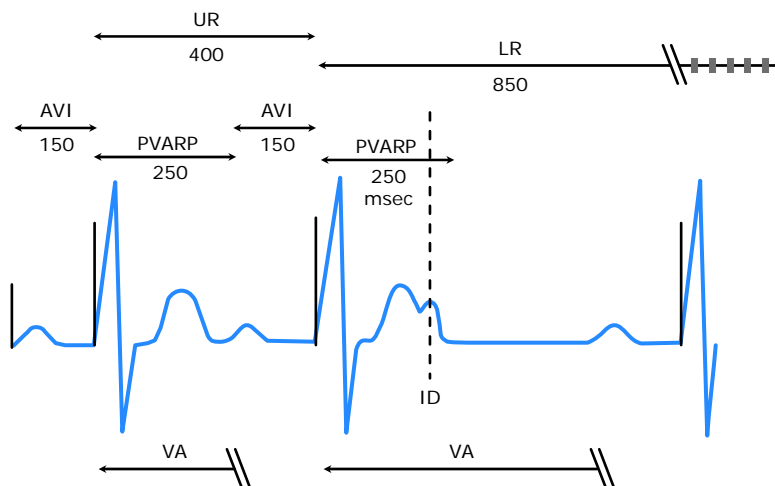


33

BIOTRONIK
excellence for life

33

Schematic of DDD Timing

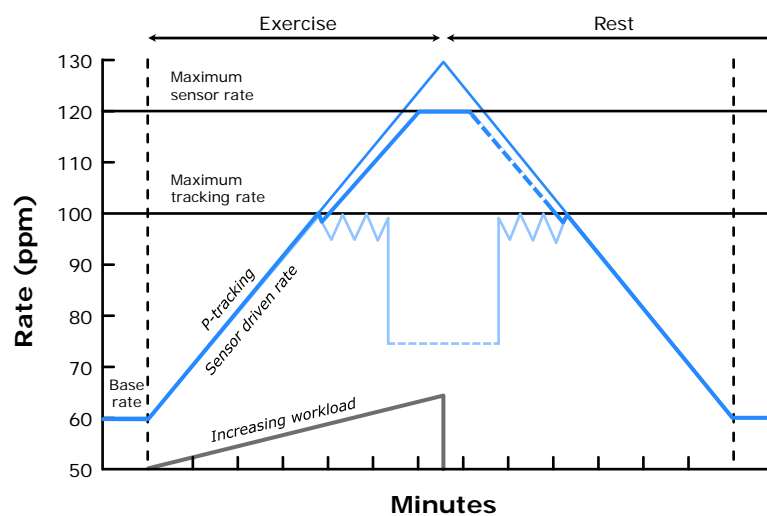


34

BIOTRONIK
excellence for life

34


Schematic of Sensor-Driven Rate Smoothing

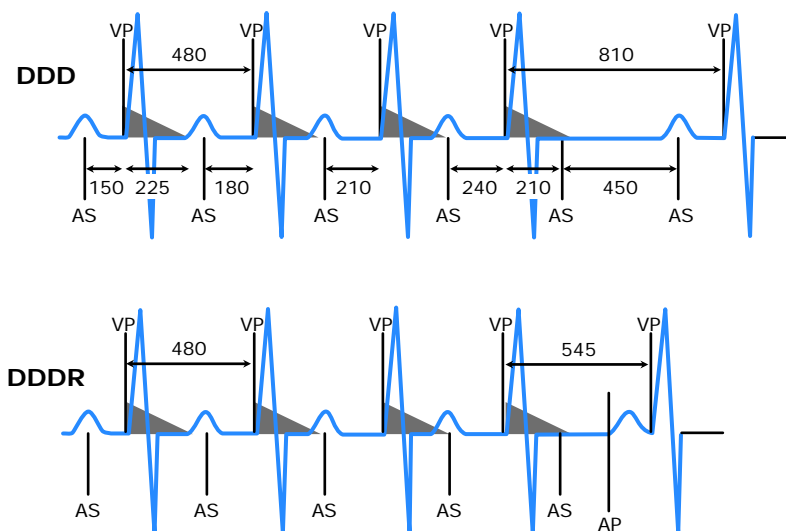

35

BIOTRONIK
excellence for life


35

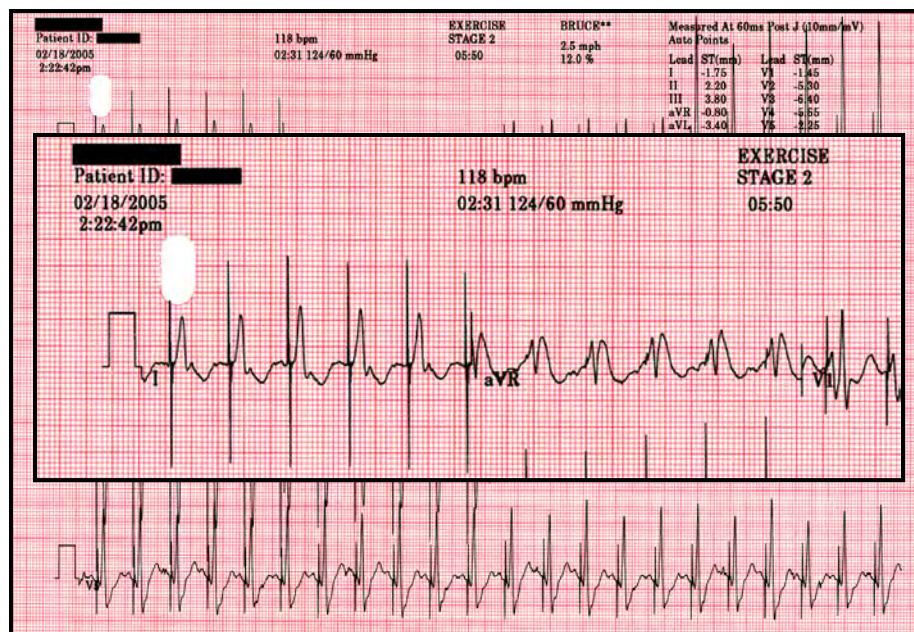


36

BIOTRONIK
excellence for life

36

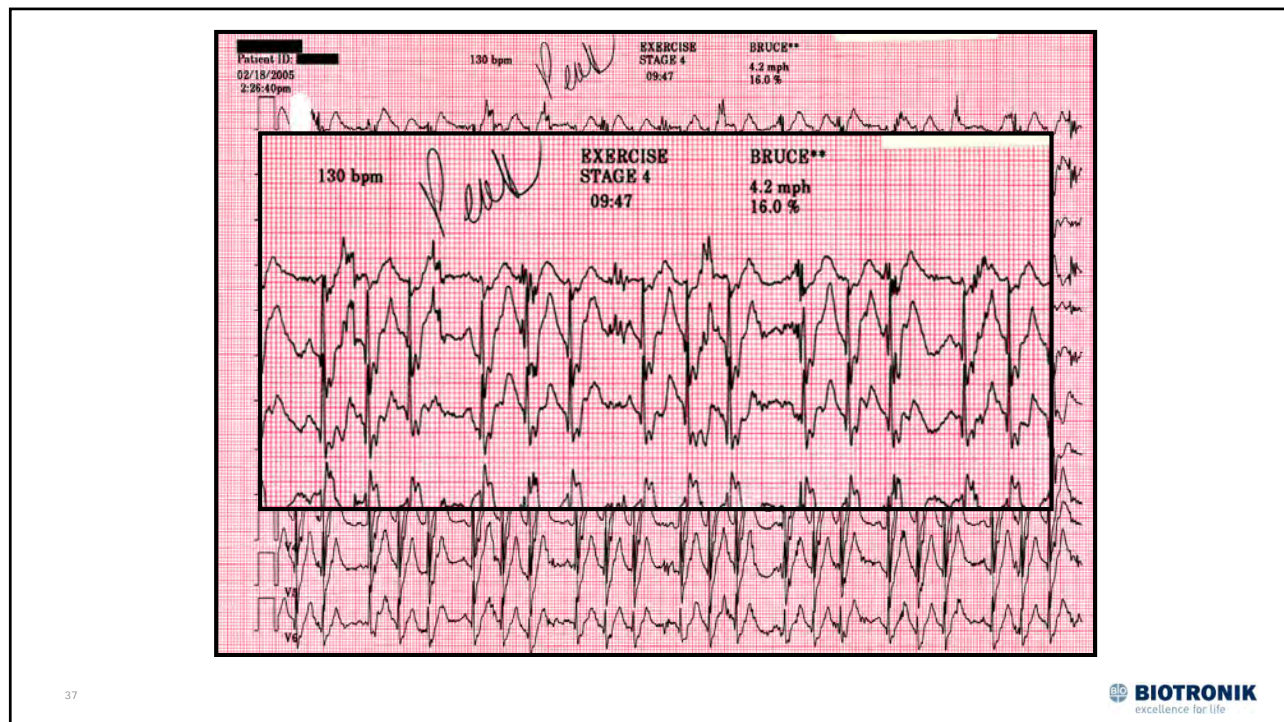

37

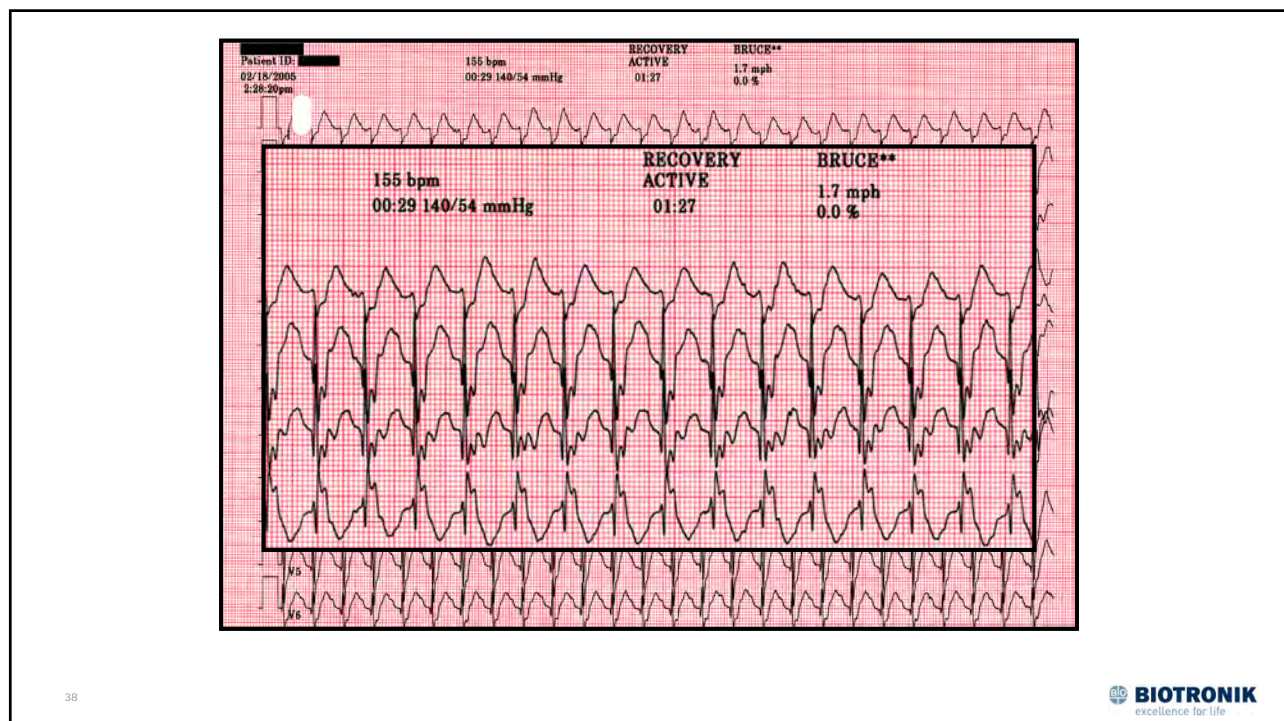

38

Timing Cycles

- Understanding basic timing cycles is critical for ECG interpretation of all CIEDs.
- Exceptions and nuances exist for every manufacturer.
- If interpretation doesn't fit basic timing cycle principles, contact technical services for assistance.